

Quick guide

How to delete previous results (Vcheck V200)




* The deletion method is the same as before.


* If you would like to delete all the results at once, you can select [Delete All]. All the results will be deleted without the need for further action.

- * ① If the number of results you would like to delete are below 100, the deletion setting unit will be displayed as 100.
- * ② For 100 ~ 2,000 results, the deletion setting unit will be the total number of results.
(e.g., ① In case, 7 results are stored → Deletion setting unit will be displayed as 100,
② In case, 999 results are stored → Deletion setting unit will be displayed as 999)
* The deletion setting unit can be set in units of 100.


* The calibration information will still be kept after deleting results.

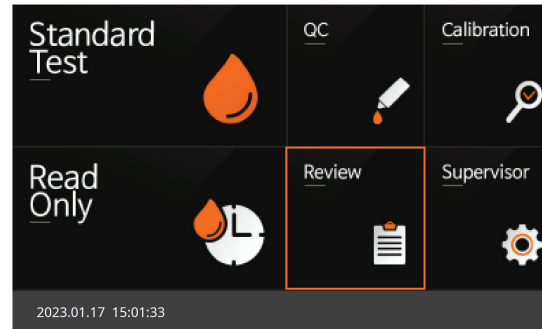


1 Turn on the Vcheck V200 and select [Review].

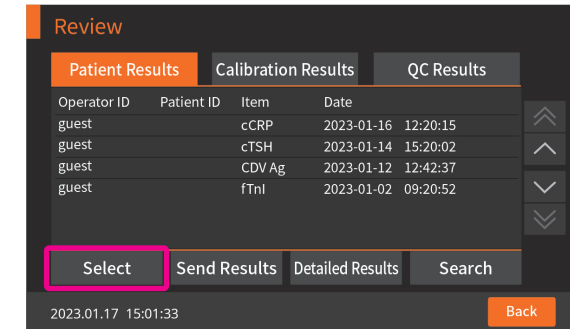


2 On the review page, click the [Select] button.

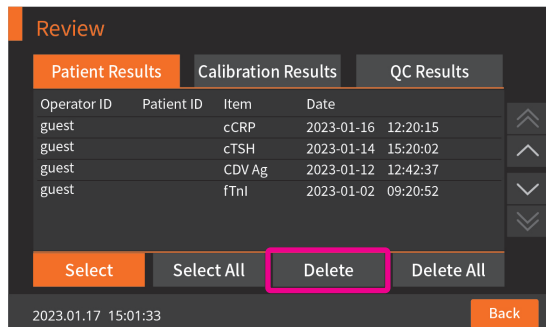

3 Click the [Delete] button.

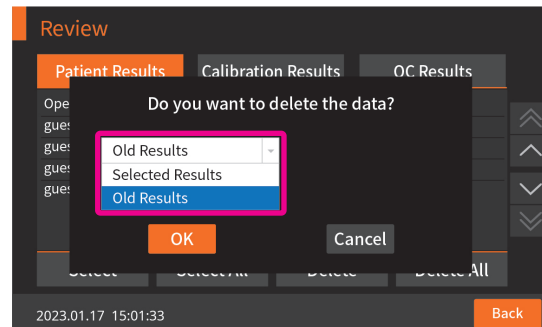

4 In the drop-down bar, select the [Old Results].

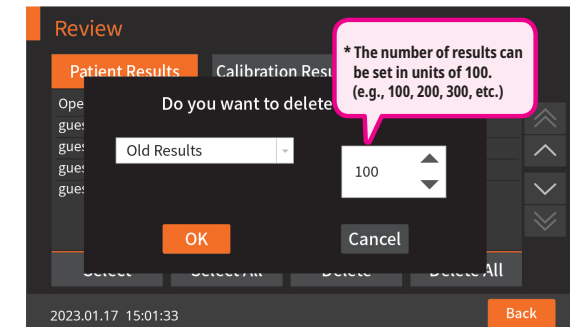

5 If you select [Old Results], you can set between a minimum of 100 up to a maximum of 2000 results to delete.