

Anigen Rapid CCV Ag

General Information

Purpose : Detection of Canine Coronavirus antigen

Principle : Immunochromatographic assay

Specimen : Canine feces

Materials provided :

- Anigen Rapid CCV test device
- Assay diluents tube
- Disposable swab
- Disposable dropper



Special Features :

- World's first CCV Ag rapid test
- One-step testing procedure : Quick & Accurate
- High sensitivity and specificity
 - Sensitivity : 93.1% vs. PCR
 - Specificity : 97.5% vs. PCR




Test Procedure

[Proper sample amount collected by a swab]

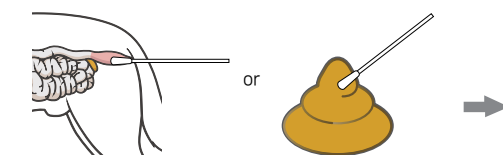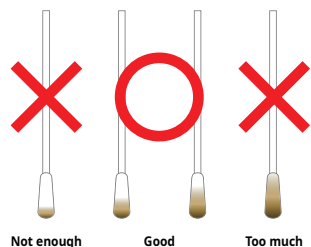


1 Collect a sample from the canine feces by using the swab

2 Insert the swab into the sample collection tube and stir the swab at least 10 times

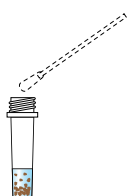

3 Discard the swab while squeezing the swab against the wall of the tube

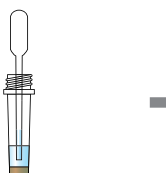


4 Wait until large particles to sink and take the supernatant by using the disposable dropper provided



5 Add 4 drops into the sample hole on the test device

5-10 min.

Interpretation

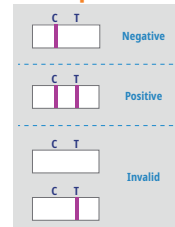

Ordering Information

Cat. No.	Description	Type	Pack size
RG1104DD	CCV Ag	Device	10 Tests/Kit