For veterinary use only

Vcheck cCortisol

FOR THE QUANTITATIVE
MEASUREMENT OF CORTISOL
CONCENTRATION IN CANINE SERUM

Intended Use

The Vcheck cCortisol is an *in vitro* diagnostic test kit for the quantitative measurement of cortisol concentration in canine serum. Cortisol is one of steroid hormone in a glucocorticoid hormone group that are important in the regulation of metabolism. Cortisol is excreted in the adrenal glands in response to stress or low blood sugar concentrations. The Vcheck cCortisol can be used to assist in the monitoring of cortisol related diseases. The Vcheck cCortisol is designed to be used only by veterinarians.

Principle

The Vcheck cCortisol Test kit is a fluorescent immunoassay for the quantitative measurement of canine cortisol concentration. The Vcheck cCortisol Test is based on the competitive immunoassay method. When the specimen is delivered to the sample hole of the test device, cortisol in the specimen and fluorescence micro particles conjugated to cortisol-BSA in conjugate pad migrates along the nitrocellulose membrane. Cortisol in the specimen competes with the Cortisol-BSA for binding sites of the anti-cortisol antibodies coated on the membrane. The density of the test line is inversely proportional to the canine cortisol concentration in canine serum. The BIONOTE Vcheck Analyzer reads the density of the test line and calculates the cortisol concentration from the calibration curve data. The control line is a reference line that indicates the test has been performed correctly.

Materials provided

	Reagent	10 Tests/Kit
①	Vcheck cCortisol Test device	10
②	Assay diluent tube	10
③	Disposable pipette tip	20
④	Instructions for use	1

Materials required, but not provided

1. BIONOTE Vcheck Analyzer
2. 50 $\mu\ell$ pipette
3. 100 $\mu\ell$ pipette

Storage and Stability

1. Store the test kit at 2–8°C. DO NOT FREEZE.
2. Do not store the test kit in the direct sunlight.
3. The test kit is stable until the expiry date that is marked on the package label.

Precautions

1. The test kit is for canine use only. Do not use for other animals.
2. The test device is sensitive to humidity and heat. Perform the test immediately after removing the test device from the foil pouch.
3. Do not reuse test components.
4. Do not touch the membrane in the result window of the test device.
5. Do not use the test kit beyond the stated expiry date marked on the label.
6. Do not use the test kit if the pouch is damaged or the seal is broken.
7. Do not mix components from different lot numbers, the components in this kit have been quality control tested as a standard batch unit.
8. All samples should be handled as being potentially infectious. Wear protective gloves while handling samples. Wash hands thoroughly afterwards.
9. Decontaminate and dispose of all samples, used kits and potentially contaminated materials safely in accordance with national and local regulations.
10. It is recommended to use a plain tube for sample collection. Test results cannot be guaranteed for anything other than the recommended tubes. Do not use SST (serum separator tube), as the gel in the tube can interfere with results.
11. Hemolyzed samples or serum with precipitate such as fibrin may give erroneous results.
12. Strictly follow the test procedure including the amount of sample used, as failure to do so may adversely affect test performance and/or produce invalid results.
13. Although the Vcheck cCortisol Test Kit offers simple and quick quantitative measurement of cortisol concentration in canine serum, there may be a difference in the detection performance with other clinical or laboratory methods with more sophisticated principles.
14. Final diagnosis must be confirmed by a veterinarian with other clinical data available.
15. BIONOTE Vcheck analyzer is recommended to use at 15–30°C.

Collection and Preparation of Sample

1. Canine serum should be used with this test. A method for preparing the serum is as follows.
2. **[Serum]** Collect the whole blood using a plain tube only. Do not use SST (serum separator tube), as the gel in the tube can interfere with results. Allow to settle for 30 minutes for blood coagulation and then centrifuge to obtain a serum supernatant.
3. If serum samples are not tested immediately, they should be refrigerated at 2–8°C and used within 7 days. For longer storage, serum can be frozen (-20°C or colder). Frozen samples should be brought to room temperature (15–30°C) prior to use.
4. Carefully prepare the sample, as a serum that has not been completely separated from blood corpuscle may cause erroneous results.
* Allow sample to reach room temperature (15–30°C) prior to testing.

Test procedure


1. Allow all kit components and sample to reach room temperature (15-30°C) prior to testing.
2. Test result can be measured by four available testing procedures, "Single test", "ACTH ST", "LDDST", or "HDDST". Users can select one of the test procedures depending on the situation.

[Single test]


- * To run this test, you will need one sample and one set of components (1 test device, 1 assay diluent tube, 2 pipette tips).

Incubate and Read

1. Turn on V200 Analyzer and select [Standard Test].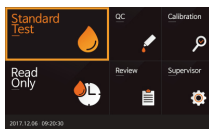


2. Type Operator ID and Patient ID or select [Direct] button.

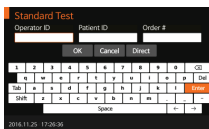
3. Remove the test device from the foil pouch. Once the "Insert Device" is displayed in the screen, insert the test device.

4. After finishing "Device Checking", select [Single Test] of the four modes displayed on the screen.

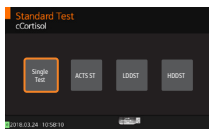
5. Use a 50 μl pipette to draw 50 μl of the sample and add to the assay diluent tube (125 μl). Then, Mix the sample with diluent by pipetting for 5–6 times.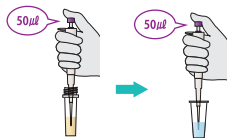


6. Select the [Incubate and Read]. Then, incubate sample mixture prepared in step “5.” for 10 minutes at room temperature (15–30°C).

* 15 seconds before the end of the incubation phase, the V200 analyzer will sound the alarm.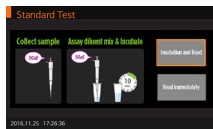

7. When [Apply sample & Start] displayed on the screen of the V200 Analyzer, draw 100 μl of incubated sample using a 100 μl pipette, add 100 μl of incubated sample into the sample hole of test device, and press [START] button immediately.

* Caution: If the time to press [START] button is delayed, it may affect the test result.

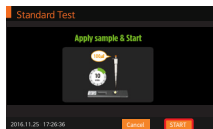
8. The V200 analyzer will display the test result on the screen after 10 minutes reading.
9. Remove the test device.

Read immediately

1. Use a 50 μl pipette to draw 50 μl of the sample and add to the assay diluent tube (125 μl). Then, Mix the sample with diluent by pipetting for 5-6 times.

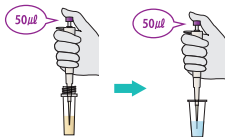
2. Incubate sample mixture prepared in step "1." for 10 minutes at room temperature (15-30°C).

3. While incubating, Turn on V200 Analyzer and select [Standard Test].


4. Type Operator ID and Patient ID or select [Direct] button.

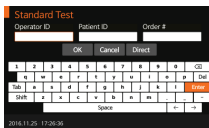5. Remove the test device from the foil pouch. Once the "Insert Device" is displayed on the screen, insert the test device.


- After finishing "Device Checking", select [Single Test] of the four modes displayed on the screen. Then, touch the [Read Immediately].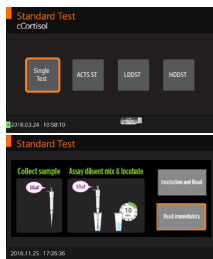

- After incubating the sample mixture for 10 minutes, add 100 μl of incubated sample into the sample hole of test device using a 100 μl pipette and press [START] button immediately.

* Caution: If the time to press [START] button is delayed, it may affect the test result.

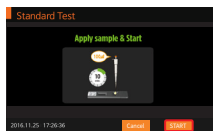- The V200 analyzer will display the test result on the screen after 10 minutes reading.
- Remove the test device.


[ACTH ST]

* To run this test, you will need two samples (Pre, Post-ACTH) and two sets of components (2 test devices, 2 assay diluent tubes, 4 pipette tips).

- Turn on V200 Analyzer and select [Standard Test].


2. Type Operator ID and Patient ID or select [Direct] button.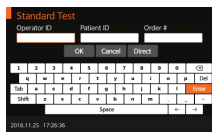


3. Remove the test device from the foil pouch. Once the "Insert Device" is displayed in the screen, insert the first test device.

4. After finishing "Device Checking", select [ACTH ST] of the four modes displayed on the screen.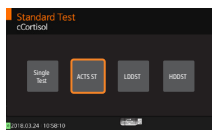


5. Use a 50 μl pipette to draw 50 μl of the Pre-ACTH sample and add to the assay diluent tube (125 μl). Then, mix the sample with diluent by pipetting for 5~6 times.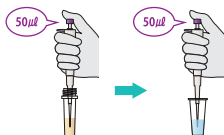


6. Select the [Incubate and Read]. Then, incubate sample mixture prepared in step "5." for 10 minutes at room temperature (15~30°C).

* 15 seconds before the end of the incubation phase, the V200 analyzer will sound the alarm.


7. After incubating the sample mixture for 10 minutes, add 100 μl of incubated sample into the sample hole of the first device using a 100 μl pipette and press [START] button immediately.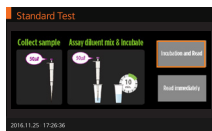

* Caution: If the time to press [START] button is delayed, it may affect the test result.

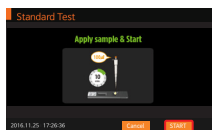
8. When [Incubate Post-ACTH sample] displayed on the screen, draw 50 μl of Post-ACTH sample using a 50 μl pipette and add it into assay diluent tube (125 μl). And then, mix the sample with diluent by pipetting for 5~6 times and incubate at room temperature (15~30°C) for 10 minutes.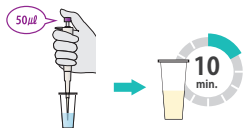


- * Caution: The figure on the screen is reading time for Pre-ACTH sample. To measure correct incubation time of Post-ACTH sample (10 minutes), it is highly recommended to use a separate timer.

9. When the test is completed, “Eject Device” will display on the screen. Then, remove the used test device and insert the new second device (Post-ACTH Device).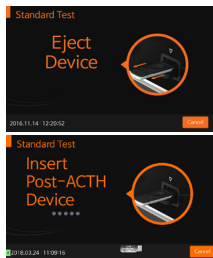

10. Using a 100 μl pipette, add 100 μl of 10 minutes incubated sample in step “8.” into the sample hole of the second device and press [START] button immediately.

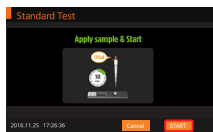- * Caution: If the time to press [START] button is delayed, it may affect the test result.

11. The V200 analyzer will display the test result on the screen after 10 minutes reading.
12. Remove the the second device.


* If you have cancelled test in the middle of above steps, please select [Standard Test] to continue or find the test result from [Review] in main menu.

[LDDST] or [HDDST]


* The test procedures of [LDDST] and [HDDST] are the same.

* To run this test, you will need three samples (Pre, Post-4H, Post-8H) and three sets of components (3 test devices, 3 assay diluent tubes, 6 pipette tips).

1. Turn on V200 Analyzer and select [Standard Test].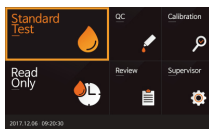


2. Type Operator ID and Patient ID or select [Direct] button.

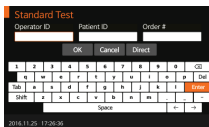
3. Remove the test device from the foil pouch. Once the "Insert Device" is displayed in the screen, insert the first test device.

4. After finishing "Device Checking", select [LDDST] or [HDDST] of the four modes displayed on the screen.

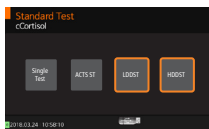
5. Use a 50 μl pipette to draw 50 μl of the pre-dexamethasone sample and add to the assay diluent tube (125 μl). Then, mix the sample with diluent by pipetting for 5–6 times.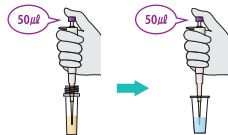


6. Select the [Incubate and Read]. Then, incubate sample mixture prepared in step "5." for 10 minutes at room temperature (15–30°C).

* 15 seconds before the end of the incubation phase, the V200 analyzer will sound the alarm.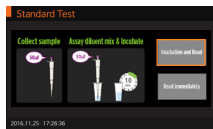


7. After incubating the sample mixture, add 100 μl of incubated sample into the sample hole of the first device using a 100 μl pipette and press [START] button immediately.

* Caution: If the time to press [START] button is delayed, it may affect the test result.

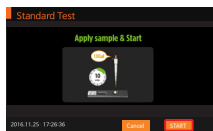
8. When [Incubate Post-4H sample] displayed on the screen, draw 50 μl of 4-hours post-dexamethasone sample using a 50 μl pipette and add it into assay diluent tube (125 μl). Mix the sample with diluent by pipetting for 5–6 times and incubate at room temperature (15–30°C) for 10 minutes.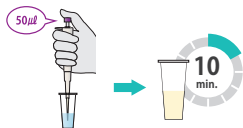


* Caution: The figure on the screen is reading time for pre-dexamethasone sample. To measure correct incubation time of Post-4H sample (10 minutes), it is highly recommended to use a separate timer.

9. When the test is completed, "Eject Device" will display on the screen. Then, remove the used test device and insert the new second device (Post-4H Device).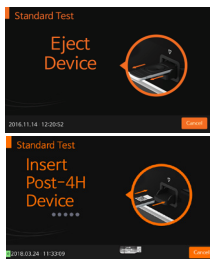


10. Using a 100 μl pipette, add 100 μl of 10 minutes incubated sample in step "8." into the sample hole of the second device and press [START] button immediately.

- * Caution: If the time to press [START] button is delayed, it may affect the test result.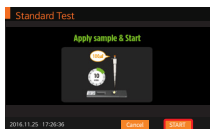


11. When [Incubate Post-8H sample] displayed on the screen, draw 50 μl of 8-hours post-dexamethasone sample using a 50 μl pipette and add it into assay diluent tube (125 μl). Mix the sample with diluent by pipetting for 5~6 times and incubate at room temperature (15~30 $^{\circ}\text{C}$) for 10 minutes.


- * Caution: The figure on the screen is reading time for Post-4H sample. To measure correct incubation time of Post-8H sample (10 minutes), it is highly recommended to use a separate timer.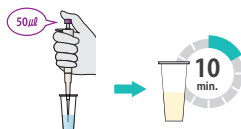

- When the test is completed, "Eject Device" will display on the screen. Then, remove the used test device and insert the new third device (Post-8H Device).

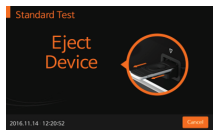
- Add 100 μl of 10 minutes incubated sample in step "11." to the sample hole of the third device using a 100 μl pipette and press [START] button immediately.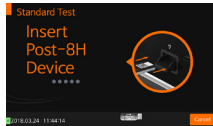

* Caution: If the time to press [START] button is delayed, it may affect the test result.

- The V200 analyzer will display the test result on the screen after 10 minutes reading.
- Remove the the third device.

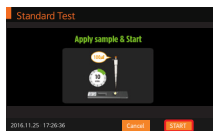* If you have cancelled test in the middle of above steps, please select [Standard Test] to continue or find the test result from [Review] in main menu.

Interpretation of the Result

- Read the concentration value of canine cortisol appearing on the display of the BIONOTE Vcheck Analyzer (1 ~ 30 $\mu\text{g}/\text{dL}$).
- If " \downarrow 1 $\mu\text{g}/\text{dL}$ " appears on the display, it means the concentration of canine cortisol in the specimen is less than 1 $\mu\text{g}/\text{dL}$.
- If " \uparrow 30 $\mu\text{g}/\text{dL}$ " appears on the display, it means the concentration of canine cortisol in the specimen is greater than 30 $\mu\text{g}/\text{dL}$.
- 1 $\mu\text{g}/\text{dL}$ is equal to 27.59 nmol/L.
- If the [Invalid] result appears on the screen, a retest shall be carried out.
- Please refer to the guide provided by the pharmaceutical manufacturer when interpreting the results for treatment monitoring purposes.

[ACTH stimulation Test]

Pre-ACTH	Interpretation	Post-ACTH	Interpretation
< 2 µg/dL	If Pre- and post-ACTH results are < 2 µg/dL, results are consistent with hypoadrenocorticism	< 2 µg/dL	If Pre- and post-ACTH results are < 2 µg/dL, consistent with hypoadrenocorticism
2-6 µg/dL	Normal	2-6 µg/dL	Equivocal
		6-18 µg/dL	Normal
		18-24 µg/dL	Equivocal
		> 24 µg/dL	Consistent with hyperadrenocorticism

[Low-Dose Dexamethasone Suppression Test]

4-hour cortisol level	8-hour cortisol level	Interpretation
-	< 1 µg/dL	Normal
1-1.4 µg/dL	1-1.4 µg/dL	Equivocal
> 1.4 µg/dL and > 50% of baseline	> 1.4 µg/dL and > 50% of baseline	Consistent with Hyperadrenocorticism
< 1.4 µg/dL or < 50% of baseline	> 1.4 µg/dL and > 50% of baseline	Consistent with PDH
> 1.4 µg/dL or > 50% of baseline	> 1.4 µg/dL and < 50% of baseline	Consistent with PDH
< 1.4 µg/dL or < 50% of baseline	> 1.4 µg/dL and < 50% of baseline	Consistent with PDH

[High-Dose Dexamethasone Suppression Test]

4-hour cortisol level	8-hour cortisol level	Interpretation
< 1.4 µg/dL or < 50% of baseline	> 1.4 µg/dL and > 50% of baseline	Consistent with PDH
> 1.4 µg/dL and > 50% of baseline	< 1.4 µg/dL or < 50% of baseline	Consistent with PDH
< 1.4 µg/dL or < 50% of baseline	< 1.4 µg/dL or < 50% of baseline	Consistent with PDH
> 1.4 µg/dL and > 50% of baseline	> 1.4 µg/dL and > 50% of baseline	Additional testing required to differentiate PDH from ATH

Specificity

The specificity of the Vcheck cCortisol was determined by studying the cross-reactivity of compounds whose chemical structure or concurrent usage may potentially interfere with the Vcheck cCortisol.

Compound	Concentration of spiking (µg/dL)	Cross (%)
Aldosterone	1000	<1 %
Corticosterone	400	5.29 %
Cortisone	400	6.12 %
Fludrocortisone acetate	1000	< 1 %
17a-Hydroxy progesterone	400	1.02 %
6a-Methyl prednisolone	200	17.94 %
Prednisone	16	1.05 %
Prednisolone	8	27.7 %
Estrone	500	< 1 %
Progesterone	400	< 1 %
21-Deoxycortisol	500	7.12 %

Screen messages and Trouble shooting

Error message	Error description
Contaminated Device	The test device is damaged or inserted improperly. Solution: Discard the test device and retest with a new test device and a new specimen.
Insufficient Sample	An insufficient amount of blood has been applied. Solution: Retest with a new test device with enough specimen, ensuring that blood is placed in to the narrow channel in the top edge of the test device.
Expired Device	The test devices are expired. Solution: Retest with a new test device that is not expired.
Temperature Error	The environmental temperature is above or below the operating range of the analyzer. Solution: Move to an area in the acceptable temperature range for the analyzer and perform the test. Do not heat or cool the analyzer artificially.
Printer Connection Fail	The communication between analyzer and barcode or printer has failed. Solution: Reconnect the analyzer and external device. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Barcode Error	
Extremely High Total Hemoglobin	The measured total hemoglobin is out of the range of 7 to 23 g/dL. Solution: This error occurs when a specimen has a total hemoglobin in the abnormal range. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Result: Invalid	The test is invalid. Solution: Retest with a new test device and a new patient specimen. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Calibration Overdue	The calibration is overdue. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Not Supported Device	A test device that is not supported by the analyzer has been loaded. Solution: Check whether the test device is manufactured by BioNote, Inc.
EEE	Internal error has occurred. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.

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