



Vcheck cProgesterone Diagnostic Algorithm

Expert's guideline for the optimal breeding times

START COLLECTING BLOOD EARLIER!

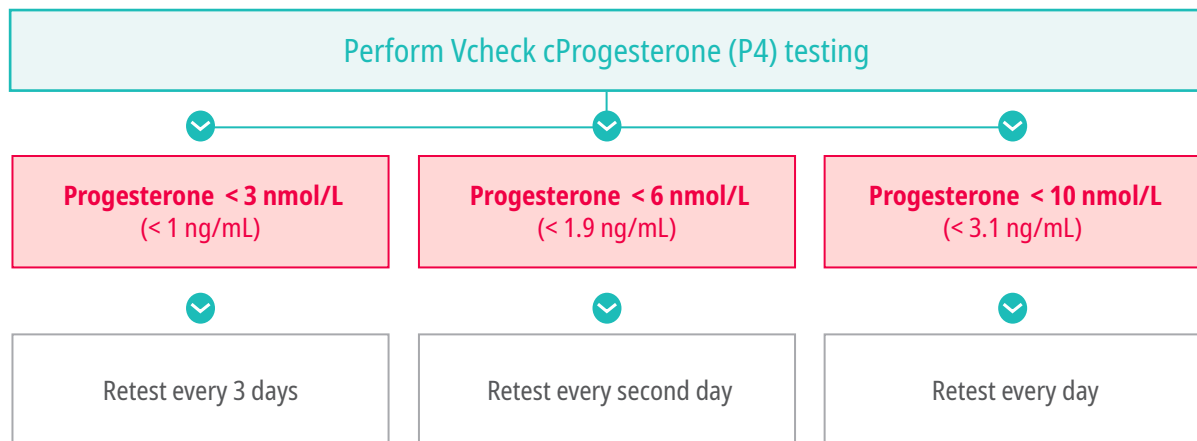


1 Natural mating

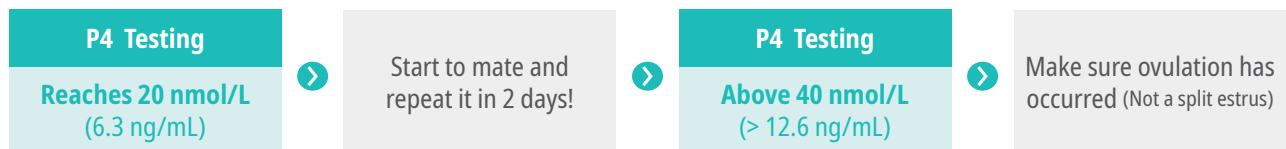

* The optimum time of fertilization: P4 19-25 nmol/L (6-8 ng/ml)

2 AI (Fresh chilled semen)

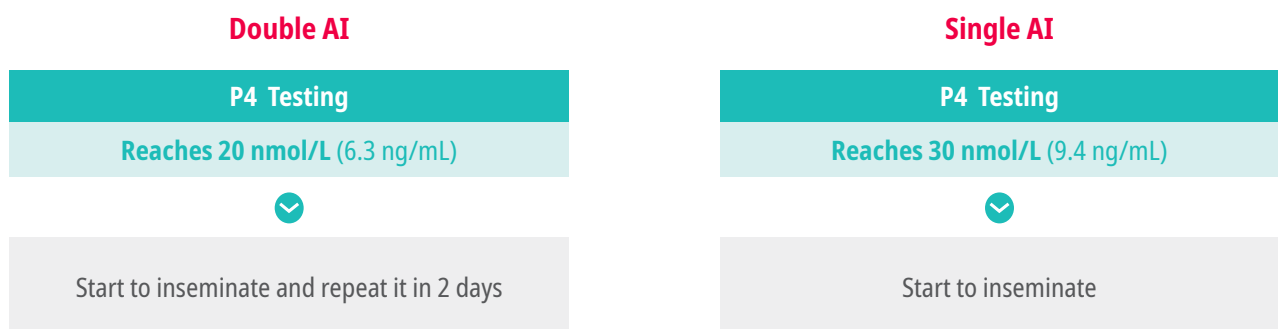


This information is a general guide only, all other symptoms and medical observations should be taken into account when making medical decisions. In no event will BioNote be liable for damages arising directly or indirectly from any use of the information contained in this document.


Vcheck cProgesterone Diagnostic Algorithm

Guidelines to predict the parturition date

1 Interpretations for spontaneous parturition¹

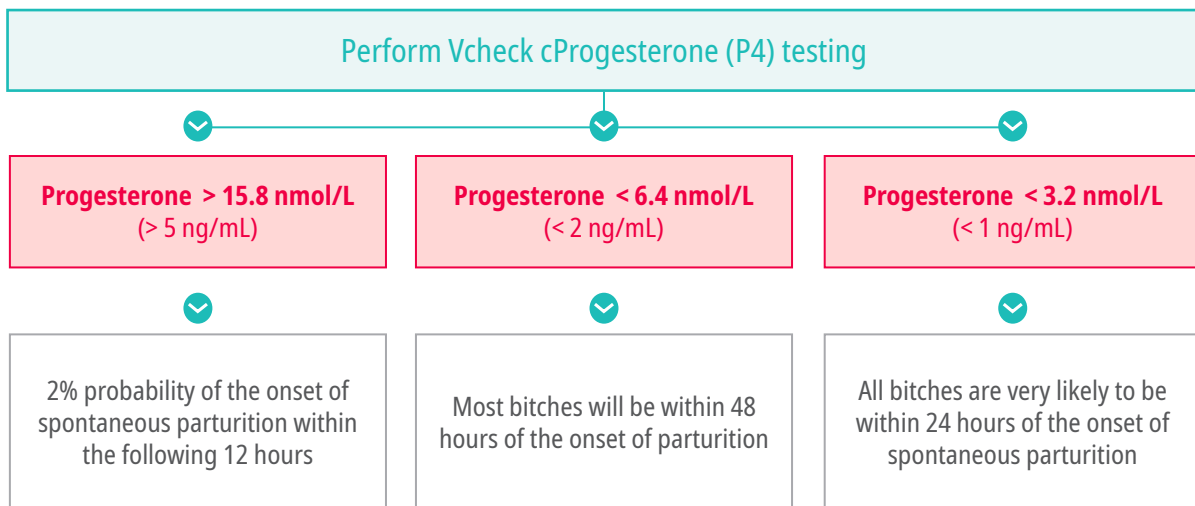

2 Interpretations for parturient C-section²

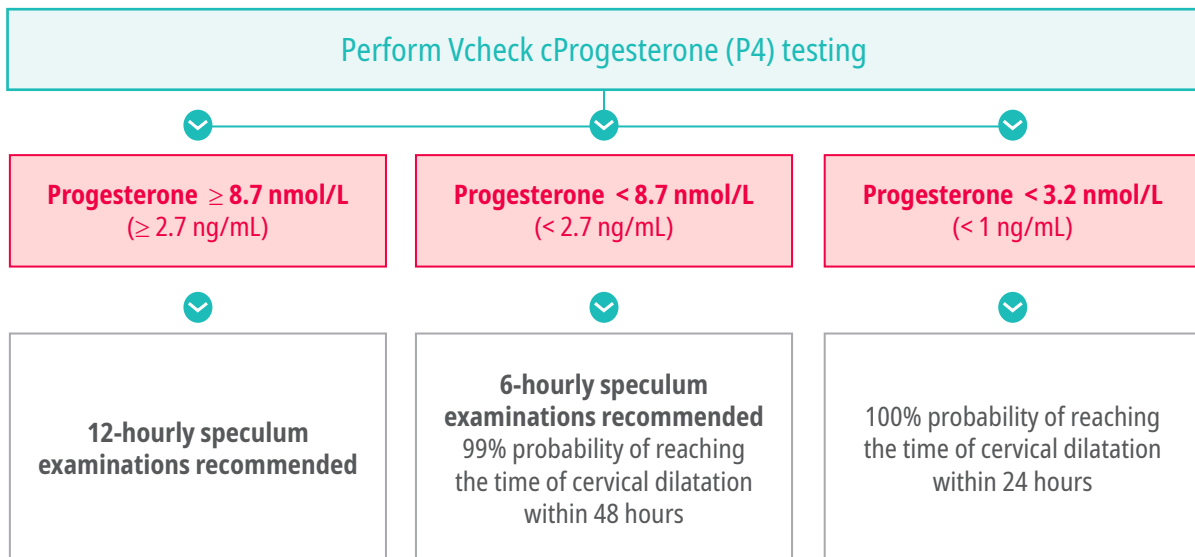

Reference

1. Dr Kurt De Cramer BVSc MMedVet (Gyn) PhD, Vet360 Issue 05 NOVEMBER 2019
2. De Cramer, K. G. M., & Nöthling, J. O. (2018). The precision of predicting the time of onset of parturition in the bitch using the level of progesterone in plasma during the preparturient period. *Theriogenology*, 107, 211–218.