INTENDED USE

The Vcheck Canine NT-proBNP is an *in vitro* diagnostic test kit for the quantitative measurement of NT-proBNP concentration in canine serum. N-terminal pro-B type natriuretic peptide (NT-proBNP) is cleaved from BNP which is produced by the muscle cells of the heart and increases with excessive stretching of the cells. NT-proBNP concentration reflects the degree of cardiac activation secondary to stimulus, such as stretching, allowing this marker to be used to assess the magnitude of cardiac muscle stretching. NT-proBNP is a valuable biomarker for differentiating cardiac and respiratory causes of dyspnea and can be used for detecting asymptomatic dilated cardiomyopathy disease in dogs. The BIONOTE Vcheck Canine NT-proBNP is designed to be used only by veterinarians.

PRINCIPLE

The Vcheck Canine NT-proBNP Test kit is a fluorescent immunoassay for the quantitative measurement of Canine NT-proBNP concentration. The Vcheck Canine NT-proBNP Test kit uses specific anti-Canine NT-proBNP antibodies that bind to Canine NT-proBNP. When the sample is delivered to the sample hole of test device, Canine NT-proBNP in the specimen reacts with the fluorescence micro particles conjugated to anti-Canine NT-proBNP antibody in the conjugate pad. The mixture then migrates along the nitrocellulose membrane and forms complexes with the anti-Canine NT-proBNP antibody coated on the membrane. The density of the test line reflects the Canine NT-proBNP concentration in the canine serum. The BIONOTE Vcheck Analyzer reads the density of the test line and calculates the NT-proBNP concentration from the calibration curve data. The control line is a reference line that indicates the test has been performed correctly.

MATERIALS PROVIDED

Reagent	5 Tests/ Kit	10 Tests/ Kit	20 Tests/ Kit
① Vcheck Canine NT-proBNP Test device	5	10	20
② Assay diluent tube	5	10	20
③ Disposable pipette tip	5	10	20
④ Instructions for use	1	1	1

MATERIALS REQUIRED, BUT NOT PROVIDED

- BIONOTE Vcheck Analyzer
- 100 µl pipette

STORAGE AND STABILITY

- Store the test kit at 2~8 °C. **DO NOT FREEZE.**
- Do not store the test kit in the direct sunlight.
- The test kit is stable until the expiry date that is marked on the package label.

Reagent	Open status	Storage	Stability	Note
Test device	Unopened	2~8 °C, Sealed	12 months	Finished product
	Opened	Do not store	-	Use directly
Assay diluent	Unopened	2~8 °C, Sealed	12 months	Finished product
	Opened	Do not store	-	Use directly

PRECAUTIONS

- The test kit is for canine use only. Do not use for other animals.
- The test device is sensitive to humidity and heat. Perform the test immediately after removing the test device from the aluminum foil pouch.
- Do not reuse test components.

- Do not touch the membrane in the result window of the test device.
- Do not use the test kit beyond the stated expiry date marked on the label.
- Do not use the test kit if the pouch is damaged or the seal is broken.
- Do not mix components from different lot numbers, the components in this kit have been quality control tested as a standard batch unit.
- All samples should be handled as being potentially infectious. Wear protective gloves while handling samples. Wash hands thoroughly afterwards.
- Decontaminate and dispose of all samples, used kits and potentially contaminated materials safely in accordance with national and local regulations.
- Severely hemolyzed samples or serum with precipitate such as fibrin may give erroneous results.
- Although the Vcheck Canine NT-proBNP Test Kit offers simple and quick quantitative measurement of NT-proBNP concentration in canine serum, there may be a difference in the detection performance with other clinical or laboratory methods with more sophisticated principles.
- Professional veterinarian should make a final diagnosis based on the results of this product, other test results and clinical findings.
- Strictly follow the test procedure (E.g. adequate sample volume), as failure to do so may adversely affect test performance and/or produce invalid results.
- BIONOTE Vcheck analyzer is recommended to use at 15~30 °C.

COLLECTION AND PREPARATION OF SAMPLE


- Canine serum should be used with this test.
- [Serum]**
 - Collect the whole blood using a plain tube **ONLY**. Leave to settle for 30 minutes for blood coagulation and then centrifuge to obtain a serum supernatant.
 - The samples should be tested immediately after collection.** If serum samples are not tested immediately, they can be frozen at -80 °C. If stored frozen at -80 °C, samples can be stored for up to 5 years.

TEST PROCEDURE

- All reagents and samples must be at room temperature (15~30 °C) before use.
- Prepare the same number of the test device, assay diluents, and pipette tips as the number of the samples.

[Coding]


- Turn on V200 Analyzer and select "Standard Test".

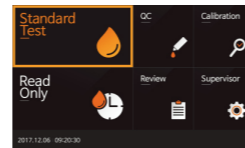- Remove the test device from the aluminum foil pouch. Once the "Insert Device" is displayed in the screen, insert the test device.

- After checking Canine NT-proBNP item name and test procedure on the display window, proceed as follows.

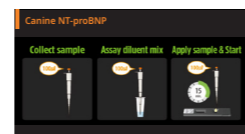
[Dilution of sample & Measurement]

- Remove the sealer of assay diluent tube, draw 100 µl sample using 100 µl pipette and add the sample into an assay diluent tube (100 µl).

- Use a 100 µl pipette to mix the sample with diluent by pipetting, for 5~6 times.

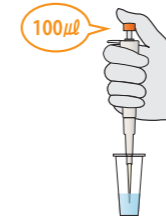
- Add the mixed sample (100 µl) into the sample hole of the test device. Then, press the "Start" button.

- * Caution: If the time to press [START] button is delayed, it may affect the test result.**

- The V200 analyzer will display the test result on the screen after 15 minutes.

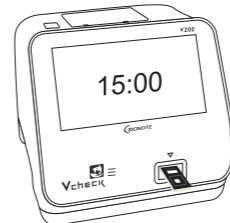- Remove the test device.

- * Strictly follow the test procedure including the amount of sample (100 µl) used and the test time (15 min), as failure to do so may adversely affect test performance and/or produce invalid results.**

INTERPRETATION OF THE RESULT

- Read the concentration value of Canine NT-proBNP appearing on the display of the BIONOTE Vcheck Analyzer. (500~10,000 pmol/L)
- If "↓ 500 pmol/L" appears on the display, it means the concentration of Canine NT-proBNP in the specimen is less than 500 pmol/L.
- If "↑ 10,000 pmol/L" appears on the display, it means the concentration of Canine NT-proBNP in the specimen is greater than 10,000 pmol/L.
- If the [Invalid] result appears on the screen, a retest shall be carried out.

REFERENCE RANGE

< 900 pmol/L	900 – 1,800 pmol/L	> 1,800 pmol/L
Normal	Suspected* Additional diagnostics are recommended	Abnormal* Additional diagnostics are recommended

- * 'Suspected' or 'Abnormal' NT-proBNP test results should always be interpreted in combination and other diagnostic findings, such as an echocardiogram.

- ** Concentration over 735 pmol/L in Doberman Pinschers indicates an increased risk for occult dilated cardiomyopathy.

PERFORMANCE CHARACTERISTIC

- Measuring Range
500~10,000 pmol/L of canine NT-proBNP is measurable in serum. For ↑10,000 pmol/L results, samples may be diluted with negative serum and measured again for a numerical result. To calculate the final canine NT-proBNP concentration, the measured result must be multiplied by the dilution factor.
- Interfering substances study
No interference was observed for each substance up to the concentration presented in the following table.

Interfering substances	Concentration
Hemoglobin	37.5 mg/dL
Intralipid	1,200 mg/dL
Bilirubin	5 mg/dL
Vitamin C	100 mg/dL

Severely hemolyzed samples can have an effect on the results due to interferences of hemoglobin.

SCREEN MESSAGES AND TROUBLE SHOOTING

[V200]

Error message	Error description
Contaminated Device	The test device is damaged or inserted improperly. Solution: Discard the test device and retest with a new test device and a new specimen.
Insufficient Sample	An insufficient amount of blood has been applied. Solution: Retest with a new test device with enough specimen, ensuring that blood is placed in to the narrow channel in the top edge of the test device.
Expired Device	The test devices are expired. Solution: Retest with a new test device that is not expired.
Temperature Error	The environmental temperature is above or below the operating range of the analyzer. Solution: Move to an area in the acceptable temperature range for the analyzer and perform the test. Do not heat or cool the analyzer artificially.
Printer Connection Fail	The communication between analyzer and barcode or printer has failed. Solution: Reconnect the analyzer and external device. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Barcode Error	The measured total hemoglobin is out of the range of 7 to 23 g/dL. Solution: This error occurs when a specimen has a total hemoglobin in the abnormal range. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Extremely High Total Hemoglobin	The test is invalid. Solution: Retest with a new test device and a new patient specimen. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Result: Invalid	The calibration is overdue. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Calibration Overdue	A test device that is not supported by the analyzer has been loaded. Solution: Check whether the test device is manufactured by BioNote, Inc.
Not Supported Device	Internal error has occurred. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
EEE	

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